

## SAFELY REMOVE YOUR DISPOSABLE GLOVES



Continue to pull the glove down and over the inside-out glove being held in your gloved hand.

Peel downwards, away from the wrist, turning the glove inside out.

With your un-gloved hand, slide your finger/s under the wrist of the remaining glove. **Do not touch** the outer surface of the glove.

Always Remember



Clean to Clean
A clean bare hand touches only
clean areas inside the other glove

Dirty to Dirty

Contaminated surfaces only touch other contaminated surfaces

- X De not touch your face or adjust PPE with contaminated gloves
- x Do not touch other items like keyboards, computer mice, door handles, etc. with contaminated gloves.
- Never wash or reuse disposable gloves

- Safely remove excess liquid beforehand
- Remove and change glove's if damaged or heavily soiled.
- Wash hands immediately after removing gloves





